

Operating the ExtriCare 3600


Canister Installation

Press the Canister into place by lining up the notches on the bottom of the pump with the tabs at the bottom of the canister. Secure the canister to the pump by lifting the canister clip and placing into the top notch of pump.



Power

The main on/off switch is on the back of the pump

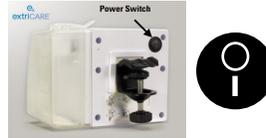

Pressure Setting Controls

The up  and down  buttons below the screen allow you to set the desired therapy pressure. Pressure can be set between 40mmHg and 200mmHg, in 5 mmHg increments.

Mode (M)

To switch between continuous and intermittent mode, press the mode button  below the screen for 2 seconds. A solid line  will appear when in continuous mode and a dashed line  will appear when in intermittent mode.

Continuous Mode operates at the set pressure without change. When Intermittent mode is selected the pump will operate at the set pressure for 5 minutes, then change to 20mmHg for 2 minutes and repeat the cycle.

Start/Stop Therapy

To start or stop therapy (suction) press and hold the start/stop button  at the lower right of the screen for 2 seconds. Although the screen will be illuminated therapy will not start until it is activated. A blue triangle  will appear at the bottom of the screen confirming therapy is activated.

Lock/Unlock Pump

The pump will automatically lock after 3 minutes.

To lock or unlock the pump hold both the and Mode (M)  Start/Stop  buttons for 2 seconds. A lock or unlock icon will appear on screen indicating its status.



Recharging the Pump

The charging port is located on the left side of the pump. Be sure to use the charger that is included with the pump. A plug icon  will appear at the top of the screen to confirm the unit is plugged in and charging. The unit will operate on battery power up to 24 hours.



346-445-4017

www.streamlinemedequip.com

