

extriCARE® 3000

Quick Start Guide




Operating the Device



To install canister: First, line up the snap hooks at the bottom of the canister of the extriCARE® 3000 device and then press canister clip into place.



Power ON/OFF: To power on the device, press the **POWER SWITCH** located on the side panel of the device **AND HOLD FOR THREE SECONDS.**

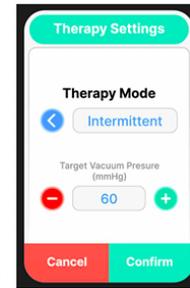


Pressure setting controls: Press the Therapy Settings Button on the bottom left corner of the main screen. Use the Green (+) and Red (-) buttons to adjust pressure. When desired pressure is reached press Green Confirm button.

Therapy Mode: To change therapy mode (Continuous or Intermittent) use the blue Arrow button to toggle to the desired setting. Press Green Confirm button to save.



To Start Treatment: Press and hold the "ex" logo button until it turns solid green.

To End Treatment: Press and hold the "ex" logo button again until the light turns off. You can also verify if therapy is active or paused by looking at the top right corner of the display screen for a play/paused symbol.

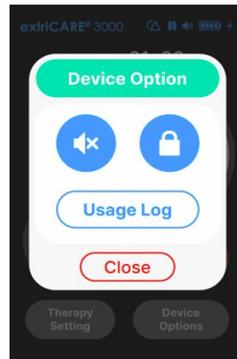


Lock/Unlock device: **To Lock:** Press the "Device Options" button on the bottom right corner of the main screen. Tap the "unlocked symbol" button. It will immediately turn blue and change to the locked symbol. Then Tap the "close" button to go back to the Main Screen.


To Unlock: Press the "Device Options" button on the bottom right corner of the main screen. Press and HOLD the "locked symbol" button until the button turns gray and the symbol changes to the unlocked symbol. **NOTE: You need to hold for 5 secs.** Then Tap the "close" button to go back to the Main Screen



Recharging device: To recharge device, use the charger that is included with the extriCARE® 3000. Charging port is located on the left side of the device (see arrow). To ensure charging, turn on the pump and make sure power adapter icon is displayed.

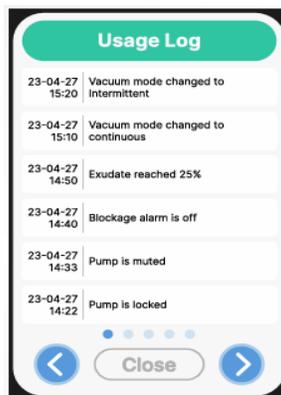

Usage Log: To access the Usage Log press the Device Options button in the bottom right corner then press Usage Log. A list of events will be displayed including: Therapy ON/OFF, Alarms, Canister Changes, etc.

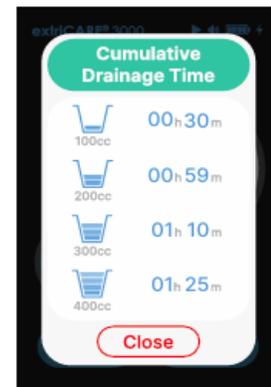

Exudate Management/Tracking: To access exudate tracking information press the Blue Canister Icon on the main screen.